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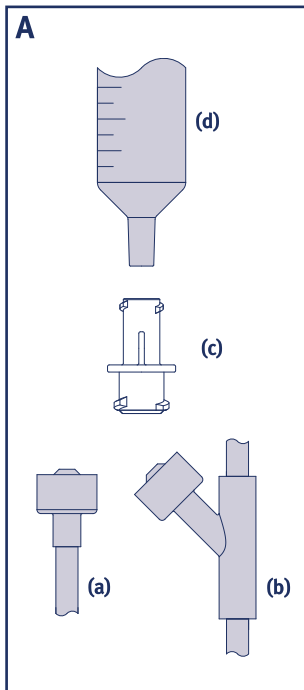
UK-IE-AU-NZ: Indications for use: This transition connector is intended for connecting an enteral male oral/Luer tip syringe to an enteral ENFit catheter/medication port. **Specifications:** DEHP-free. Natural rubber latex-free. **Duration of use:** Do not use for more than 24 hours. For single patient use. **Important notes:** For enteral use only. Ⓢ Not for intravenous / intravascular use. Ⓢ Do not use if package is damaged. **Please note:** The connector may become a source of infection after use. Follow good clinical practice and all local and national regulations for disposal.

Cedic s.r.l. Via Liberazione 63/9, 20068 Peschiera Borromeo (MI), Italy

ENFit is a registered trademark of GEDSA used with their permission

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UK-IE-AU-NZ: Instructions for use: 1. Wash hands and adopt the relevant good hygiene practice. 2. Remove the protective cap (if applicable) from the enteral ENFit catheter (a)/medication port (b). 3. Securely screw the transition connector (c) on the ENFit catheter/medication port. 4. Remove the protective cap (if applicable) from the enteral male oral/Luer tip syringe (d). 5. Securely screw or fit (as applicable) the enteral oral/Luer tip syringe into the transition connector. 6. Follow the instructions for use of devices connected with the transition connector. 7. Discard the transition connector when delivery through syringe is complete in any case within a maximum of 24 hours.