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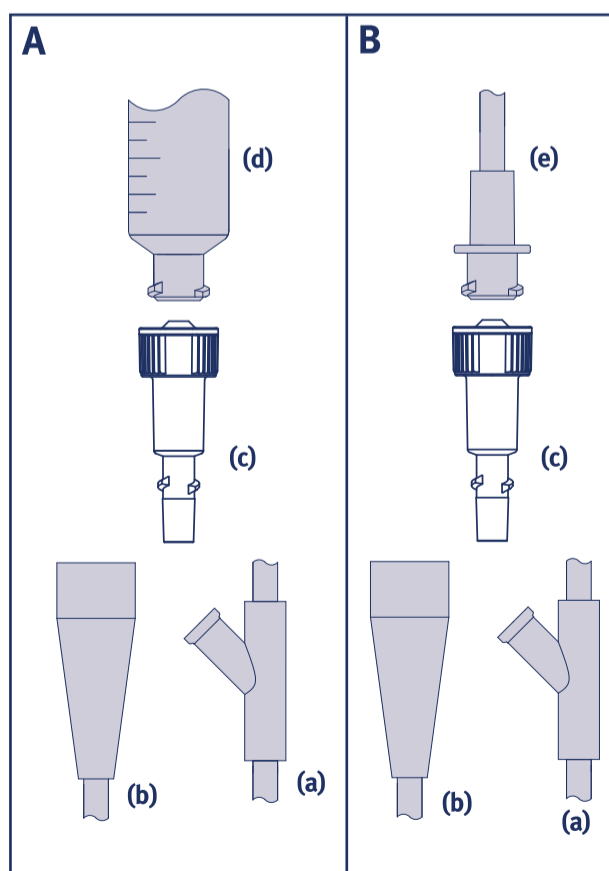
UK-IE-AU-NZ: Indications for use: This transition connector is intended for connecting an enteral ENFit giving set or an enteral ENFit tip syringe to an enteral device oral / ENLock female connector or an enteral catheter soft funnel. **Specifications:** DEHP-free. Natural rubber latex-free. **Duration of use:** Do not use for more than 24 hours. For single patient use. **Important notes:** For enteral use only. Ⓢ Not for intravenous / intravascular use. Ⓢ Do not use if package is damaged. **Please note:** The connector may become a source of infection after use. Follow good clinical practice and all local and national regulations for disposal.

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UK-IE-AU-NZ: Instructions for use: 1. Wash hands and adopt the relevant good hygiene practice.

A (Enteral ENFit tip syringe use): 2. Remove the protective cap (if applicable) from the enteral device oral (a) / ENLock (b) female connector or from the enteral catheter soft funnel (b). 3. Securely screw or fit (as applicable) the transition connector (c) into the oral / ENLock female connector or into the soft funnel. 4. Remove the protective cap (if applicable) from the enteral ENFit tip syringe (d). 5. Securely screw the enteral ENFit tip syringe into the transition connector. 6. Follow the instructions for use of devices connected with the transition connector. 7. Discard the transition connector when delivery through syringe is complete in any case within a maximum of 24 hours.

B (Enteral ENFit giving set use): 2. Remove the protective cap (if applicable) from the enteral catheter oral (a) / ENLock (b) female connector or from the enteral catheter soft funnel (b). 3. Securely screw or fit (as applicable) the transition connector (c) into the oral / ENLock female connector or into the soft funnel. 4. Remove the protective cap (if applicable) from the enteral ENFit giving set (e). 5. Securely screw the enteral ENFit giving set into the transition connector. 6. Follow the instructions for use of devices connected with the transition connector. 7. Discard the transition connector when delivery through the enteral giving set is complete in any case within a maximum of 24 hours.